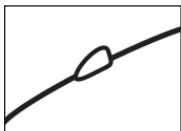
1

2

3

4

5

6

7

8

## Deutsch

Liebe Eltern!

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## English

Dear Parents,

Please read the instructions for use carefully as they contain important information which you should bear in mind when using this medical device. If you have any questions, please consult your doctor or pharmacist.

### Use of the NYDA® lice comb

What are you looking for?

**Head lice:** Grey to brownish, the size of a sesame seed. ①

**Nymphs:** Small, recently hatched lice. ②

**Nits:** White to brownish, oval lice eggs, the size of a grain of sand. Unlike dandruff, they stick firmly to hair. ③

### Diagnosis:

Inspect all scalp hair of each family member for head lice and nymphs with the NYDA® lice comb according to the instructions for use.

### Treatment:

If infestation with head lice is discovered, it is essential to start treatment with an effective head lice product, e.g. NYDA® against lice and nits. NYDA® is highly effective and well tolerated. After observing the first application period of NYDA® the dead lice and nits can be combed out with the lice comb.

### Brief information on the head lice product NYDA®

NYDA® is based on dimeticone.

NYDA® is used for the physical treatment of scalp hair in case of infestations with head lice (pediculosis capitis).

NYDA® creeps deeply into the breathing system of lice, nymphs and eggs and displaces the oxygen and finally seals the breathing openings. This leads to the suffocation of all developmental stages. If used as instructed, NYDA® kills not only adult head lice but also juvenile forms (nymphs) and the eggs (nits).

### Use of the NYDA® lice comb:

1. Brush the hair carefully to remove any tangles. ④
2. Then part the hair into several sections and divide each section into fine strands. ⑤
3. Important: Hold the comb in your hand with the angled grip pointing away from the head. This is the best way of combing the lice and nits out of the hair. ⑥
4. Comb each strand individually. Comb slowly from the roots to the tips of the hair; start as close to the scalp as possible. ⑦
5. Check the comb for lice and nits after each strand and clean it carefully with a paper tissue. ⑧

Clean the NYDA® lice comb after each use with lukewarm water and soap. Do not cook! Do not wash in the dishwasher! Do not use if color or shape differ. You can dispose of the NYDA® lice comb in the household waste.

**Note:** Any nits still remaining in the hair after the complete treatment with NYDA® head lice product are usually dead and no longer contagious.

Please report all serious incidents directly to the manufacturer and the responsible national authority.

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G. Pohl-Boskamp GmbH & Co. KG